

# PERFORMING A RESCUE



Is patient responsive?



Call emergency services!



Get AED.



Place AED on ground.  
Open lid.



Remove clothing.  
Expose chest.



Attach pads.



Do not touch the patient.



If required, AED will deliver  
shock automatically.



If present, place CPR device.



Follow AED prompts.  
Give compressions.



If instructed, give breaths.

**Automatic AED**



Cardiac Science and the Shielded Heart logo are trademarks  
of Cardiac Science Corporation.

Copyright © 2012 All Rights Reserved.

N7 W22025 Johnson Drive, Waukesha, WI 53186 USA  
[www.cardiacscience.com](http://www.cardiacscience.com)



MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany



70-01092-01r E

